

---

---

# **X-ray equipment maintenance and repairs workbook**

for  
radiographers &  
radiological technologists

by  
Ian R McClelland  
Chief technical support engineer (retired)



Diagnostic Imaging and Laboratory Technology  
Essential Health Technologies  
Health Technology and Pharmaceuticals  
**WORLD HEALTH ORGANIZATION**  
Geneva

---

WHO Library Cataloguing-in-Publication Data

McClelland, Ian R.

X-ray equipment maintenance and repairs workbook for radiographers and radiological technologists / Ian R. McClelland.

1.X-rays 2.Radiography 3.Technology, Radiologic 4.Maintenance—methods  
5.Problems and exercises I.Title.

ISBN 92 4 159163 3

(NLM classification: WN 150)

© World Health Organization 2004

All rights reserved. Publications of the World Health Organization can be obtained from Marketing and Dissemination, World Health Organization, 20 Avenue Appia, 1211 Geneva 27, Switzerland (tel: +41 22 791 2476; fax: +41 22 791 4857; email: [bookorders@who.int](mailto:bookorders@who.int)). Requests for permission to reproduce or translate WHO publications—whether for sale or for noncommercial distribution—should be addressed to Publications, at the above address (fax: +41 22 791 4806; email: [permissions@who.int](mailto:permissions@who.int)).

The designations employed and the presentation of the material in this publication do not imply the expression of any opinion whatsoever on the part of the World Health Organization concerning the legal status of any country, territory, city or area or of its authorities, or concerning the delimitation of its frontiers or boundaries. Dotted lines on maps represent approximate border lines for which there may not yet be full agreement.

The mention of specific companies or of certain manufacturers' products does not imply that they are endorsed or recommended by the World Health Organization in preference to others of a similar nature that are not mentioned. Errors and omissions excepted, the names of proprietary products are distinguished by initial capital letters.

The World Health Organization does not warrant that the information contained in this publication is complete and correct and shall not be liable for any damages incurred as a result of its use.

The named authors alone are responsible for the views expressed in this publication.

Designed by minimum graphics, New Zealand  
Typeset by SNP Best-set Typesetter Ltd., Hong Kong  
Printed by Interprint, Malta

# Contents

Introductory remarks	v
Acknowledgements	vi
<b>Part I. Introduction</b>	<b>1</b>
Introduction	3
Questionnaire—Student's own department	6
Pre test	8
<b>Part II. Routine maintenance modules</b>	<b>11</b>
Module 1.0 Routine maintenance overview	13
Task 1. Maintenance survey for an X-ray room	17
Module 1.1 X-ray generator maintenance, fixed installation	19
Task 2. X-ray control familiarization. Part 1	24
Task 3. X-ray control familiarization. Part 2	26
Task 4. Test for X-ray tube overload calibration. Part 1	28
Task 5. Test for X-ray tube overload calibration. Part 2	31
Module 1.2 X-ray generator maintenance, mobile unit	32
Module 1.3 X-ray generator maintenance, C D mobile	37
Module 1.4 X-ray generator maintenance, portable unit	41
Module 2.0 X-ray tube stand maintenance	44
Task 6. X-ray tube-stand maintenance	47
Module 2.1 X-ray tube maintenance	48
Module 2.2 Collimator maintenance	50
Task 7. X-ray tube and collimator maintenance	52
Module 3.0 Bucky table & vertical Bucky maintenance	53
Module 3.1 Tomography attachment maintenance	55
Module 4.0 Fluoroscopy table maintenance	57
Module 4.1 Fluoroscopy TV maintenance	60
<b>Part III. Fault diagnosis and repair modules</b>	<b>63</b>
Module 5.0 Common procedures, for fault diagnosis and repairs	65
Task 8. Fuse identification	70
Module 6.0 X-ray generator repairs, fixed installation	71
Task 9. No Preparation, Part 1	86
Task 10. No Preparation. Part 2	87
Task 11. No Exposure	88
Task 12. X-ray output linearity	89
Module 6.1 Mobile or portable-generator repairs	90
Module 6.2 C D mobile repairs	94
Module 7.0 X-ray tube stand repairs	99
Task 13. Bucky tabletop and tube-stand centre	103

## X-RAY EQUIPMENT MAINTENANCE AND REPAIRS WORKBOOK

iv

Module 7.1	X-ray tube repairs	104
Module 7.2	Collimator repairs	110
	Task 14. Help! No spare globe for the collimator	116
Module 7.3	High-tension cable repairs	117
Module 8.0	Bucky and Bucky table repairs	121
	Task 15. A film exhibits grid lines	126
Module 8.1	Tomography attachment repairs	127
Module 9.0	Fluoroscopy table repairs	130
Module 9.1	Fluoroscopy TV repairs	135
Module 10.0	Automatic exposure control, operation and problems	140
<b>Part IV. Automatic film processor</b>		<b>145</b>
Module 11.0	Automatic film processor, routine maintenance	147
Module 11.1	Automatic film processor repairs	151
	Task 16. Films appear too dark	156
	Task 17. Films exhibit symptoms of low fixer	157
Module 11.2	The Film ID printer	158
<b>Part V. Appendices</b>		<b>161</b>
Appendix A.	Sensitometry	163
Appendix B.	Recommended tools and test equipment	169
Appendix C.	Graphs, check sheets and record sheets	177
Appendix D.	Routine maintenance check sheets	186
Appendix E.	X-ray equipment operation	205
Appendix F.	Teaching techniques	248
Appendix G.	Health and safety	253
<b>Part VI. Post test and glossary</b>		<b>257</b>
	Post test	259
	Glossary	262

## **Introductory remarks**

This document, which is developed by the International Society of Radiographers and Radiological Technologists (ISRRT) under the umbrella of the WHO Global Steering Group for Education and Training in Diagnostic Imaging, is the second in a series targeting technical aspects, including quality control of diagnostic imaging services. The document is primarily aiming at assisting radiographers and radiological technologists working in small and mid-size hospitals where resources often are limited, to optimize and improve diagnostic imaging, and to ensure the best possible use of resources according to local needs.

The document can be obtained by contacting the following address:

Team of Diagnostic Imaging and Laboratory Technology (DIL)  
World Health Organization  
20, Avenue Appia  
CH-1211 Geneva 27  
Switzerland

Fax: +41 22 7914836  
e-mail: [ingolfsdotting@who.int](mailto:ingolfsdotting@who.int)

**Harald Ostensen, MD**

Geneva, July 2004

---

## **Acknowledgements**

For their considerable input and assistance in producing this workbook, special thanks are due to:

Peter J Lloyd, Peter Hayward, Brett Richards, Sue Salthouse, Peter K Mutua, M Jean Harvey, Leonie Munro, Martin K West, Jiro Takashima, and to Graham English.